



Step 1 : Download “HeartBook” APP on Google Play or Apple Store, and have it installed on your phone.

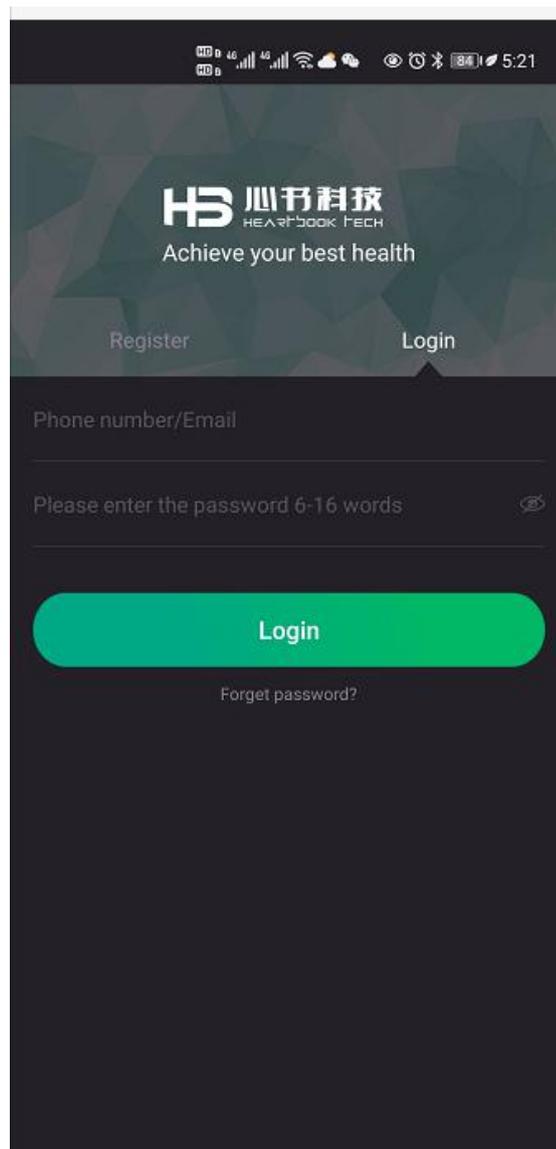

Step 2 : Please register by your personal phone number or email, and choose the correct “Country/Region”. Save the your passwords well.


After you register successfully, then log in the APP.

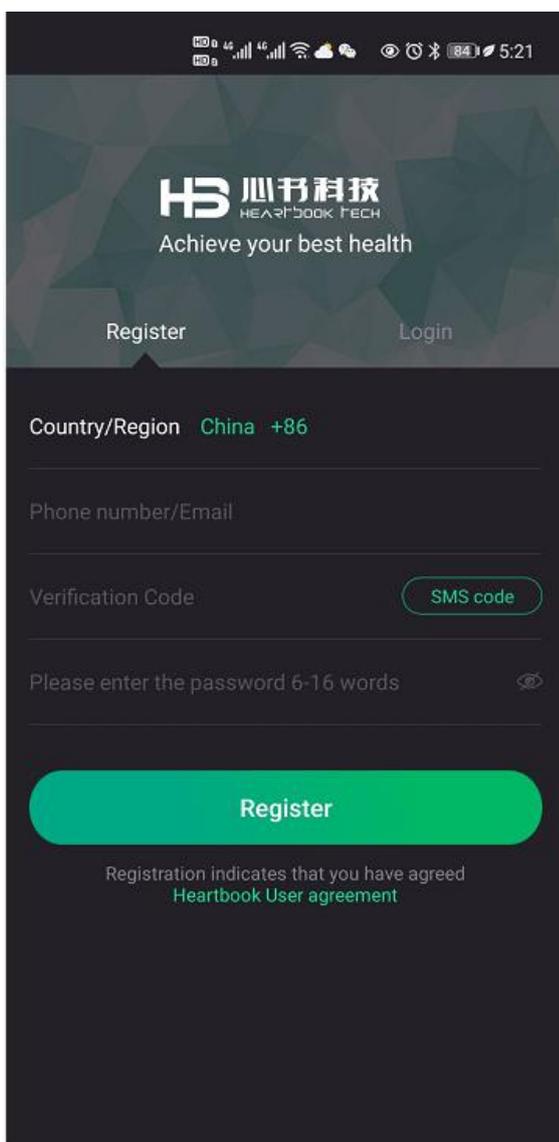

Step 3: Long press (for 3 seconds) the power button on the back of the device until the blue light is on.




Step 4: Keep the device closer to a cell phone with Bluetooth enabled, and

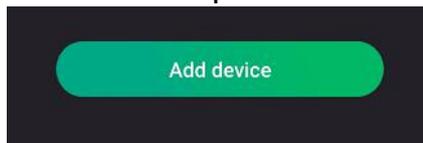

turn on the APP, then tap

to scan the device automatically.

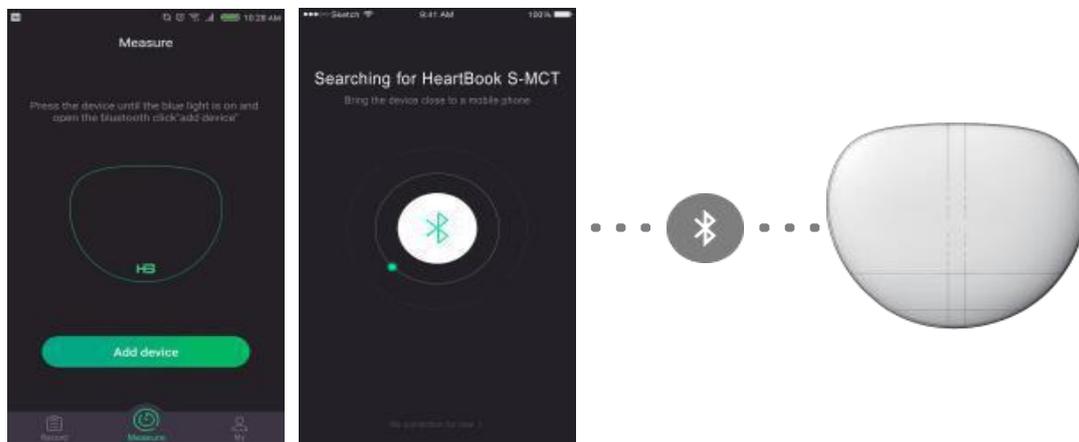

Remark: If the scan fails, tap "manual connection" to select nearby devices.




Step 5 : After getting connected successfully, please follow the illustrations for proper electrode assembly and wear as per below figures.




After assembly:



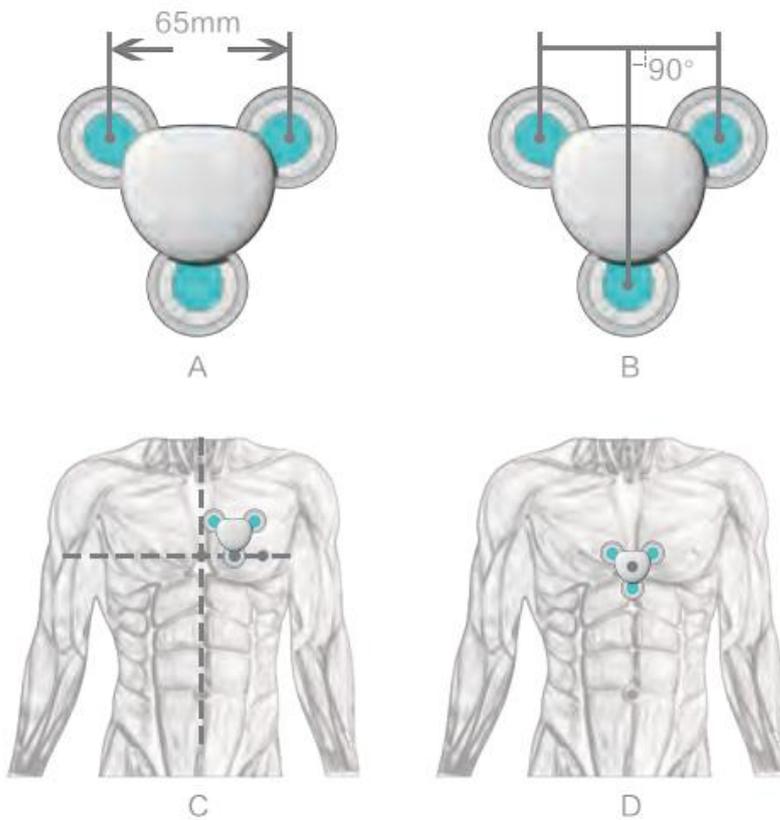



Step 5 : Enter the measurement interface, tap

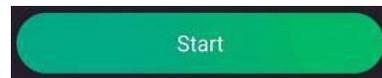



to start measurement, and tap "Stop"




to end the measurement.

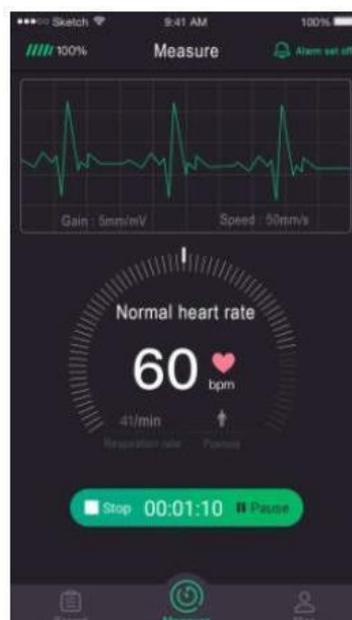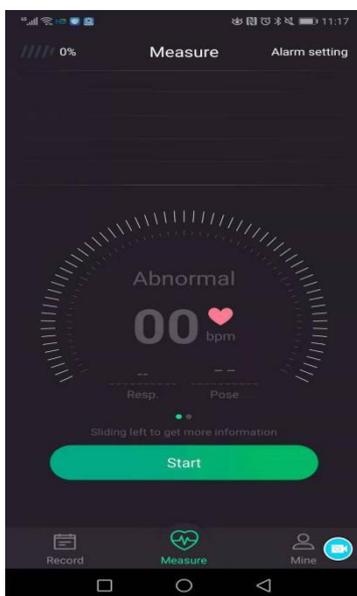


Step 6 : After you choose to stop the measurement, get HeartBook report by steps of : [Records](#) \implies [choose the date](#) \implies [Report](#) \implies [Check the report](#)

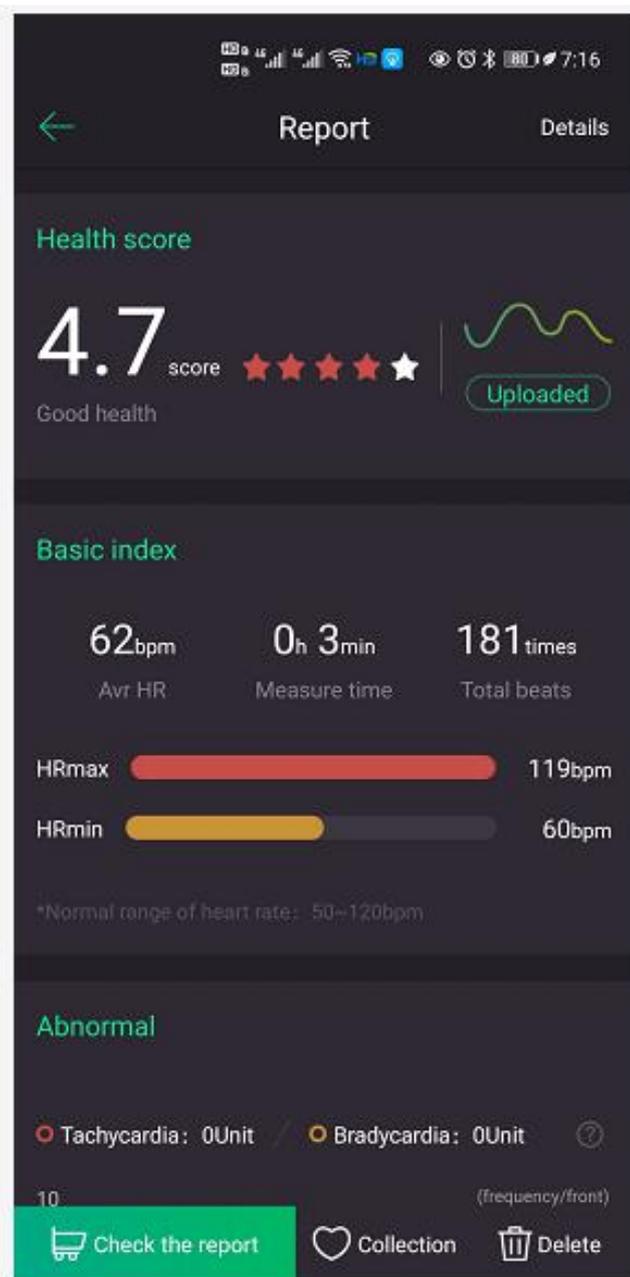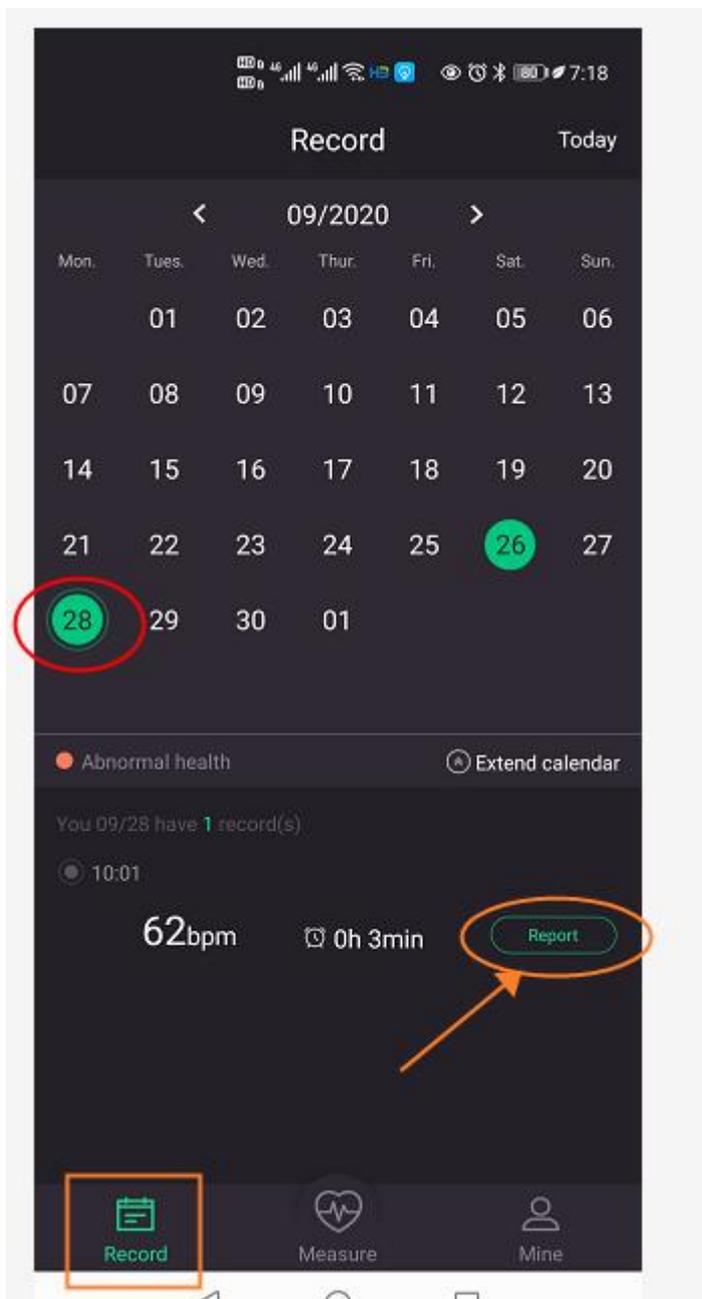

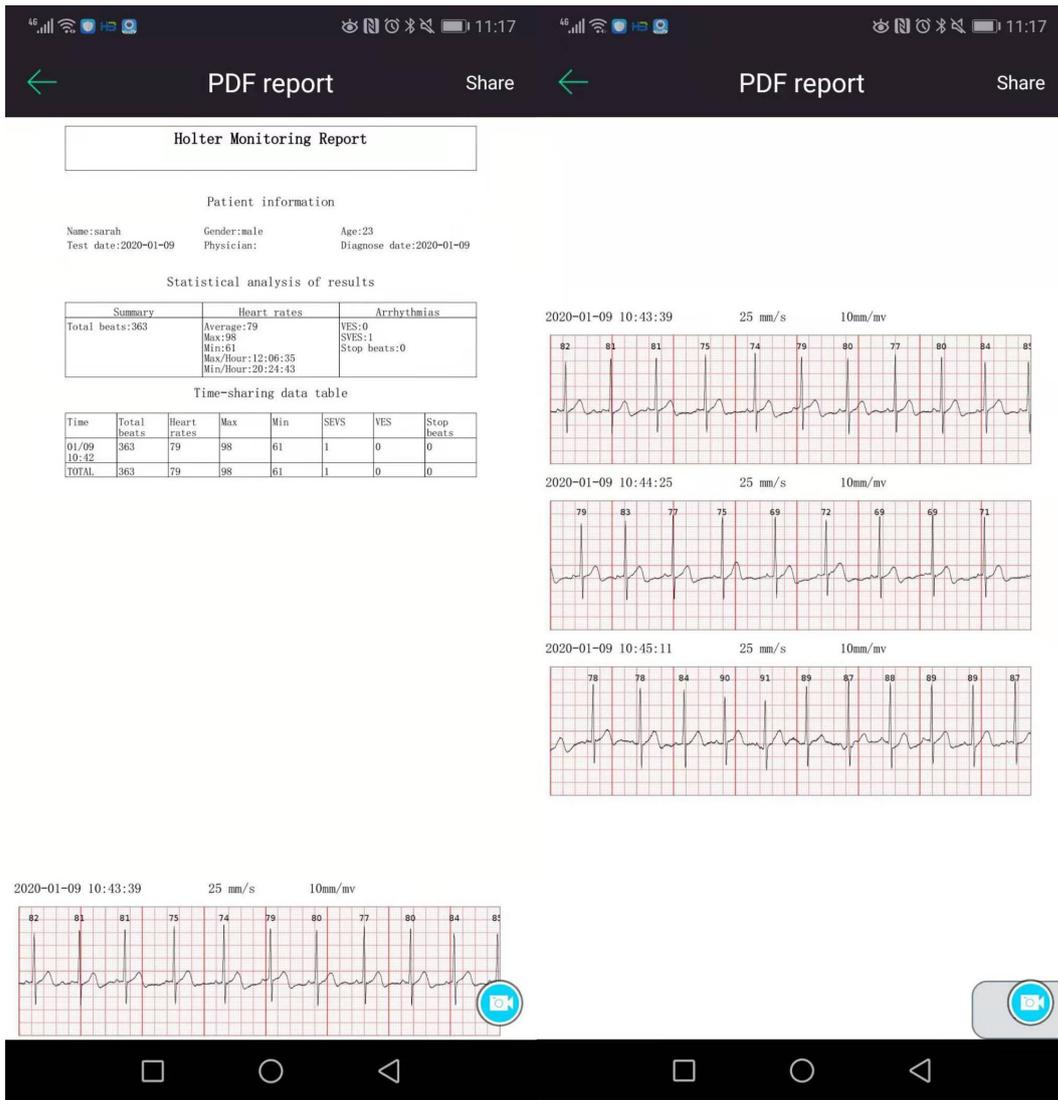

Step 7: If you want to see the electrocardiogram, please tap the “Details” on the report interface and you will see as below:

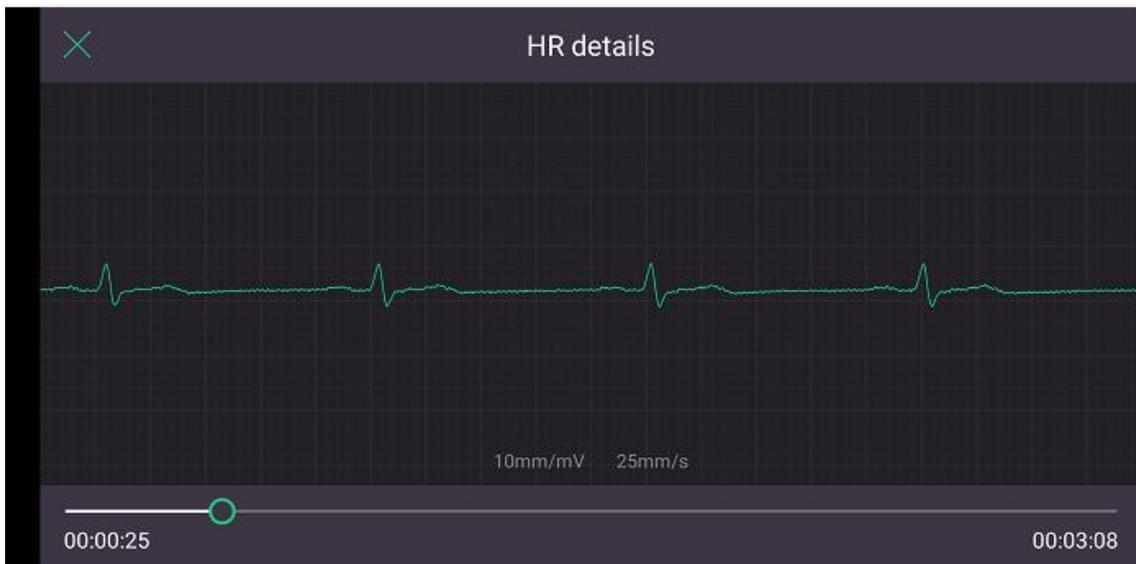

Step 8 : Share the PDF report -- Tap **Share** to share the PDF report by Whatsapp, Email or other media tools you have.

