


## Step 1. Download Stethoscope Application

iOS/Android

Application name: Smartho


App supported language: Chinese, English

<https://play.google.com/store/search?q=Smartho>

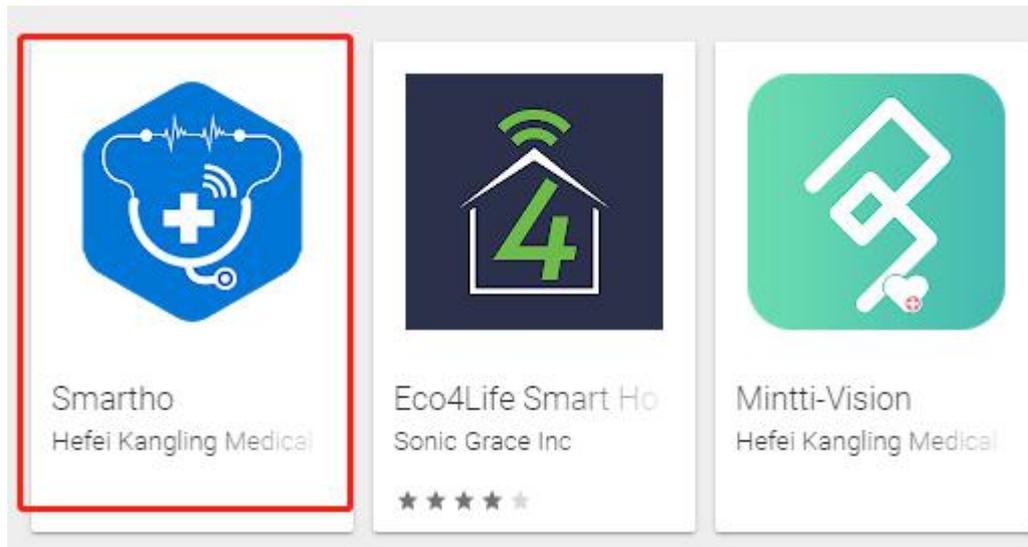

Search “Smartho” APP on Google Play Store with mobile phone and download

## Step 2. Register by phone or Email

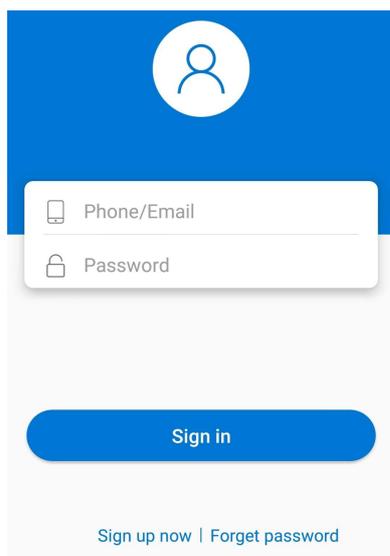

You can register the Application by Email or by phone number

### Step 3. Turn on the stethoscope

Long press (for 3 seconds) the power button.



### Step 4. Setting

Enter the menu interface

Choose the Mode: “Bell” or “Diaphragm”



Bell mode, which is suitable for listening to the heart sounds.

Diaphragm mode, which is suitable for listening to lung sounds.




## Step 5. Log in and connect

Log in the APP with your account, allow Smartho to enable Bluetooth.

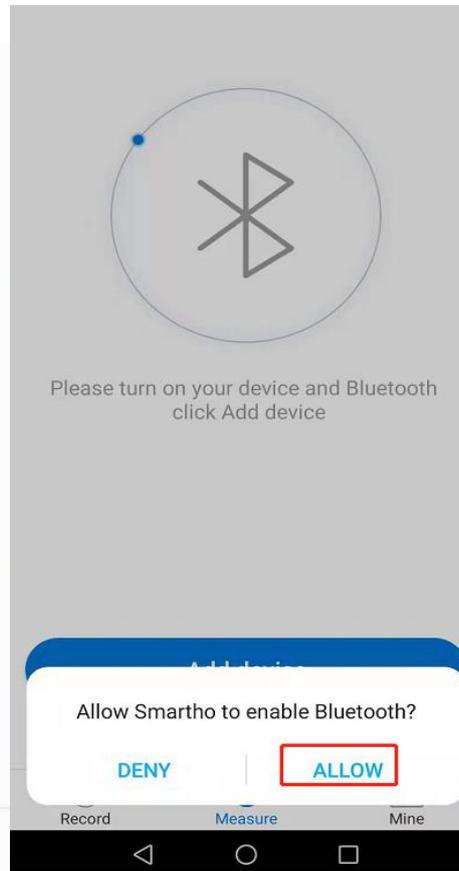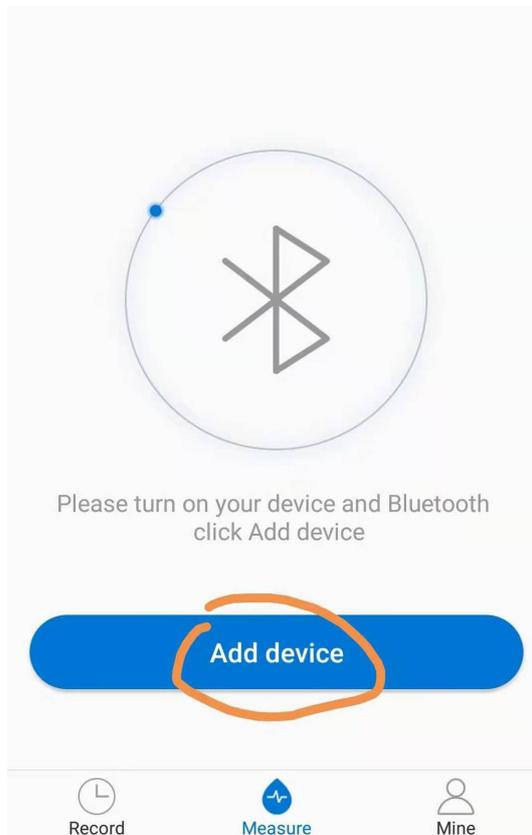

Select a connection device

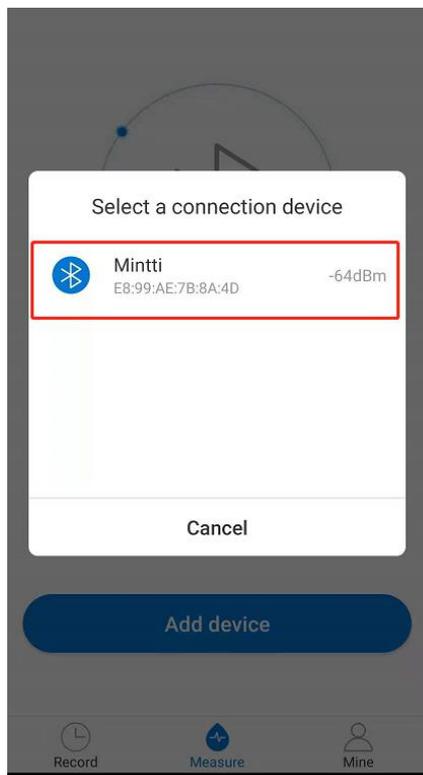


## Step 6. Measurement


Plug the headphones we provided into your mobile phone (with APP)




Tap Start measuring and save the records

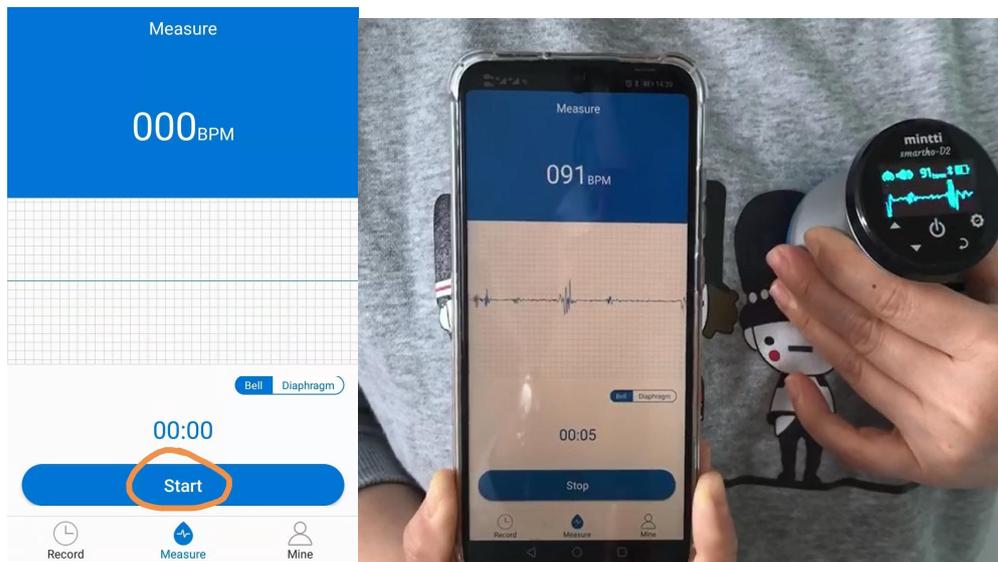

Save records

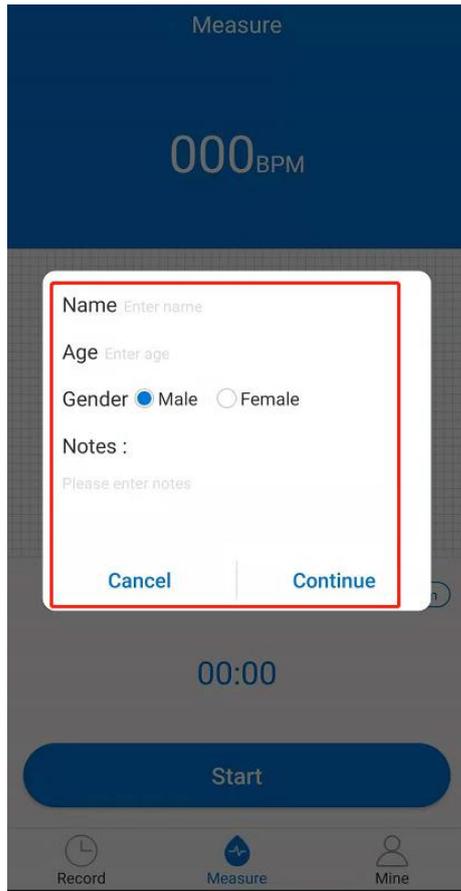


Step 7. Check the records and upload

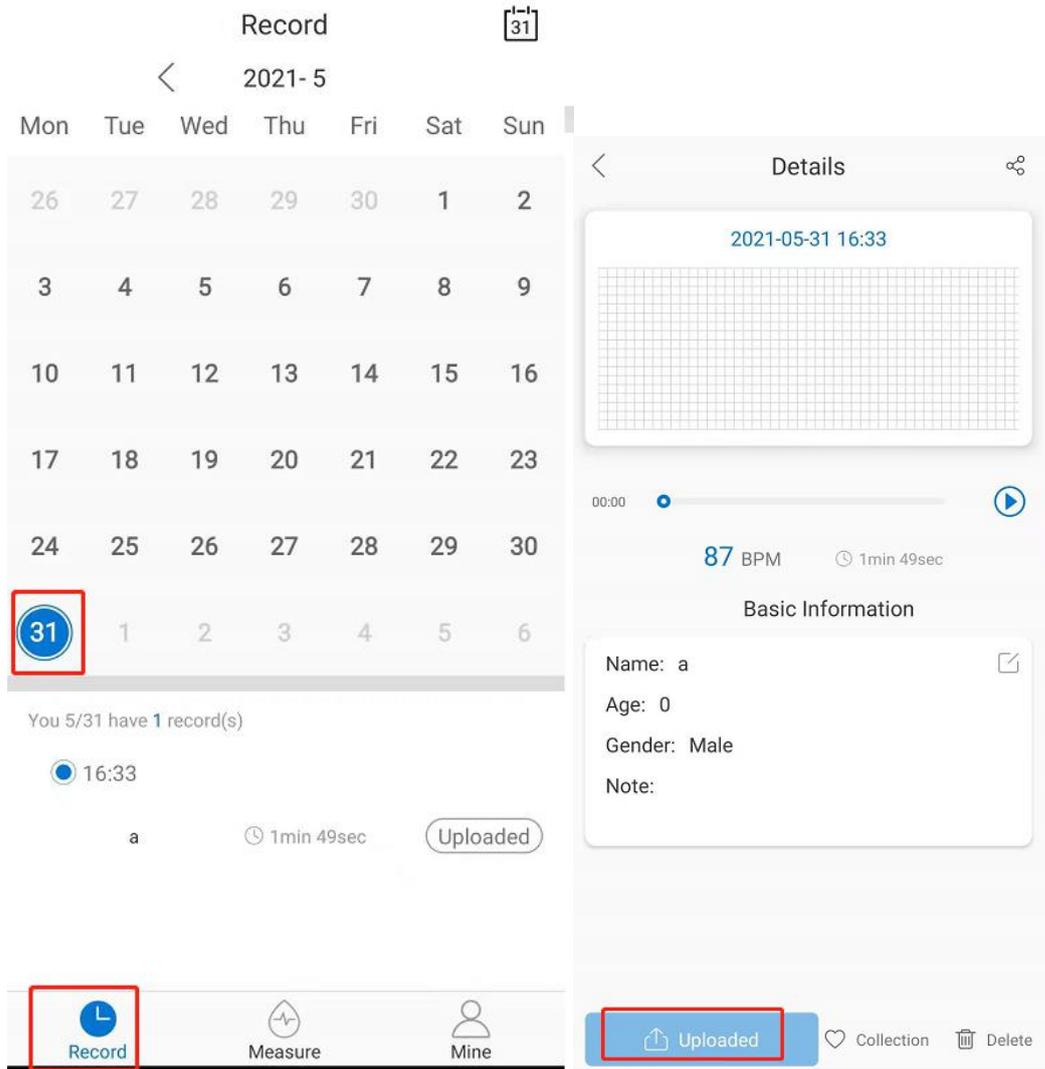

## Step 8. Share the records

You can share the records by Whatsapp, Email, Skype and so on

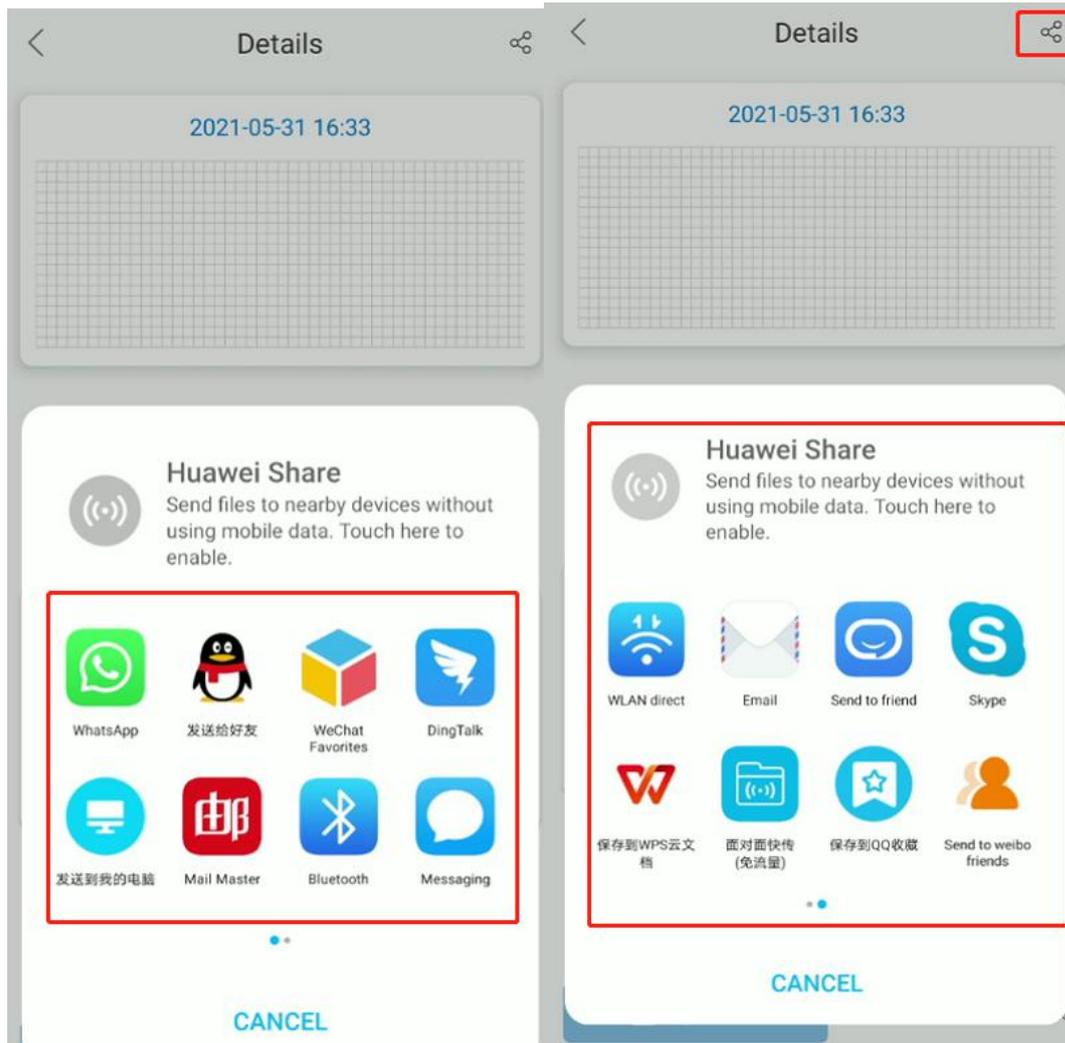

Step 9. Select the records

Files - Mintti- Wav

← Phone



Phone > Mintti > data > 2021-04-16...



Sort by time ▼



2021-04-16\_16-55-40.wav

16/04/2021 - 1.44 MB



spk\_2021-04-16\_16-55-40.pcm

16/04/2021 - 1.44 MB



raw\_2021-04-16\_16-55-40.pcm

16/04/2021 - 1.44 MB



mic\_2021-04-16\_16-55-40.pcm

16/04/2021 - 1.44 MB